

bebird *uwa*
See the best in you



Please refer to the instructions before using this product

Download App



iOS Search "Bebird"
in App store
Android Search "Bebird"
in Google Play store

or



Scan QR code to
download the App




Watch Tutorial Video
via this QR code

Auto Connection



Manual Connection

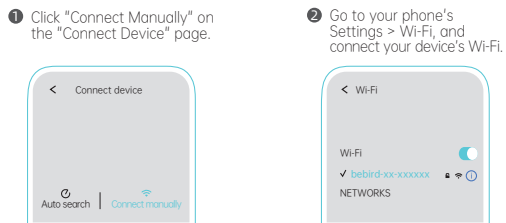

Device Overview

Silicone Covered Tips Camera Lens

Hose



Charging Port

Power Button
When the device is powered on,
a blue light will illuminate.
During power-on and connection,
the blue indicator light will flash.
Once connected, the blue light
will remain lit.
While charging, the indicator
light will glow red.
When charging is complete,
the red indicator will turn off.
The blue indicator will flash
rapidly when the battery is low.

Accessory Installation

Protective Plug :
Place the ear canal
protective plug in the
appropriate position
on the hose to prevent
it from being inserted
too deeply into the
ear canal.




Ear Spoon:
Unscrew the current
accessory by twisting it,
counterclockwise to
loosen and remove it.
Attach the desired
accessory by screwing
it in clockwise until its
tight and secure.



How to Use the Tweezers Components?

Place tweezers compent-A onto the device
body firstly, then install and tighen tweezers
component-B

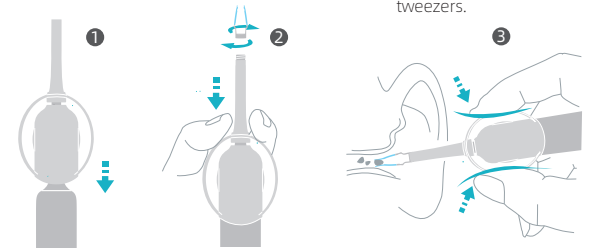

Gently pinch the handle
of the tweezers with two
fingers to close the
tweezers.

Warm Reminder
Install the tweezers component correctly before use to avoid its
loosening or falling off.


How to Use the Hose lens?

In most cases, you don't need to bend
the hose to use it. If necessary,
please note that:

Facing the scale, slowly bend the hose
with your fingers to the **left** and **right**
sides of the scale.


Do not bend in other directions, otherwise
it may cause damage to the product.

Do not use brute force.
Before using the tweezers, the bent hose
needs to **be straightened again**.

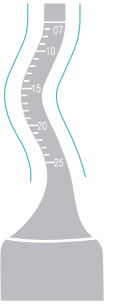


How to Use the Sticky Head or Sponge Head?

Attach the Sticky Head or Sponge Head to the hose securely, prevent loosening.
Ear debris can be Stubbord Ear debris can be removed by Sticky Head.
Oily earwax can be absorbed by the Sponge Head.

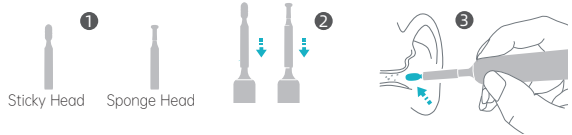


Warm Reminder : Both types of head are disposable.

How to Use the Dropper?

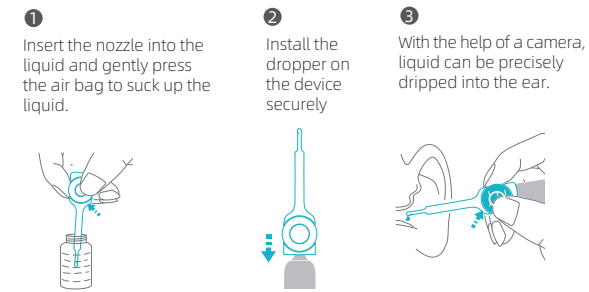


Warm Reminder:
1. Please avoid immersing the lens in liquid, otherwise it may cause the lens
to become blurred and need to be wiped before continuing to use.
2. This dropper is for use with ear drops only (this product does not contain
ear drops). If used with other liquids, please please seek professional advice.

Accessory List 1

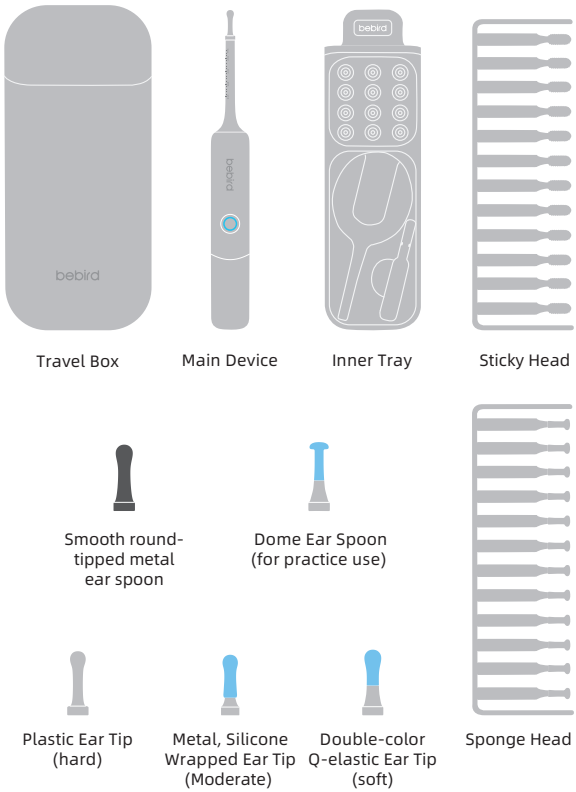

Accessory List 2

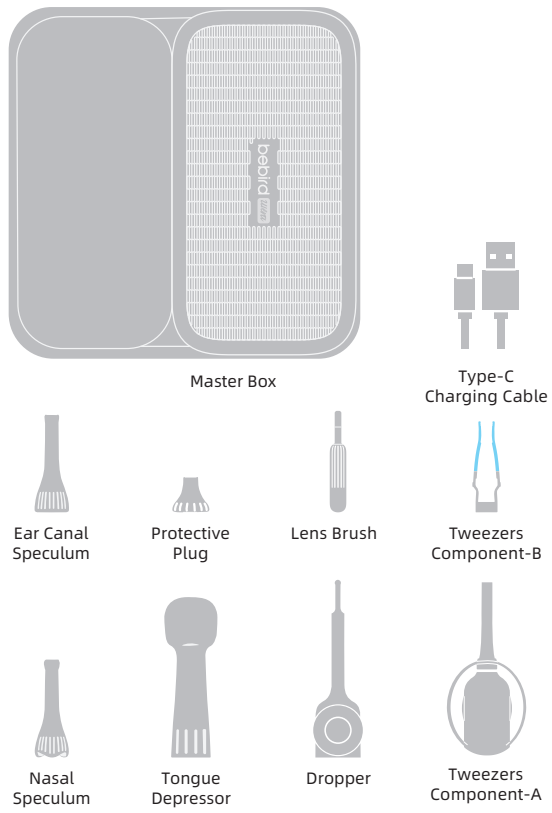

How to Use the Ear/Nasal/Throat Observation tools?



***Note:** Please make sure the tool is securely attached. If feel uncomfortable,
please do not continue.

Troubleshooting

Symptoms	Solution
Camera light off	Check if the brightness setting in the settings is set to the lowest setting.
Lens blurry	Gently wipe the lens surface with the included lense brush.
App crashes	Uninstall the existing app and reinstall.
No response on startup	Please charge it with the included charging cable.
Indicator light flashes rapidly	Battery is low, please recharge the battery in time.
The product fails to display live video after successfully connecting to the app	Check if the app permissions are disabled.
Wi-Fi connection is unstable, sometimes disconnecting within seconds	Restart the device and reconnect to the app.
Indicator light and lens light flash simultaneously	Flashing indicates a firmware upgrade to be done. Firmware upgrade will take about 30mins, please charge the device for 30 mins to avoid power cut during the upgrade. Once upgrade is done, no more flashing, and ready for normal use.
Incorrect orientation during use	Switch left/right ear mode on the bottom of the app.

If the issue persists after following the above operations, please contact our customer service, and we will provide you with further effective solutions.

Product Specifications

Brand: Bebird
Name: Bebird EarSight Ultra
Usage time: Up to 80 minutes
Image sensor: CMOS
Optimum Focal Length: 12.5mm
Intelligent orientation: Gimbaled Intelligent Gyroscope
Lens grade: 4P* Slim Lens, IP67 Waterproof,
Operating Environment: -10°C to 50°C
Image Frame Rate: ≥ 25fps
Input Current: 5V ± 0.5A

Warnings

- Cleaning:** Wipe the lens with a lens brush.
- Assembly:** Ensure all device accessories are securely attached to prevent them from falling off during use.
- Operating Environment:** Use the device in a suitable, private environment. Avoid using it in crowded spaces.
- Battery Maintenance:** The device includes a built-in rechargeable battery. If unused for extended periods, charge it once a month to maintain battery health.
- Supervision for Children:** Children should not use this device unsupervised to avoid accidental injury.
- Sunlight Exposure:** Keep the main device and accessories out of direct sunlight to prevent softening or damage.
- Water Exposure:** Do not submerge the device in any liquids.
- Temperature Notice:** During use, the device may warm slightly due to the LED lights; this is normal and is capped at approximately 45°C (113°F).
- Intended Use for Tweezers:** Use the tweezers only to remove earwax or other foreign objects from the ear canal.
- The inside of the ear canal is relatively fragile. To avoid damaging the ear canal, please do not use this product to scrape the ear canal vigorously.

Warranty Service

Customer information

Customer Name: Customer Telephone:

Retailer Name: Customer Address: