

bebird

Smart Visual Ear Cleaner
Ear cleaning for better ear protection



Recommended charging equipment
with 5V output voltage.

How to connect the App

Note:
Do not use the device unless you have successfully connected the device to your mobile phone through the Android / iOS App and are streaming live video footage.


1
Download the APP
iOS: Search for "bebird" in App Store
Android: Search for "bebird" in Google Play






OR scan the QR code
to download the app



2
Open the cover
Power on automatically

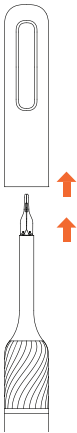

3
Open the APP
Connect device WiFi

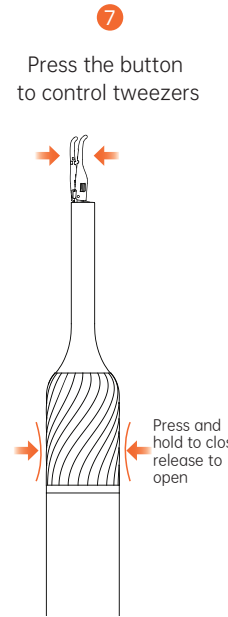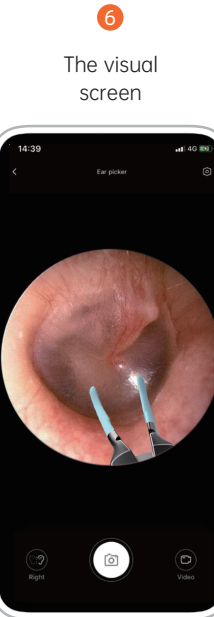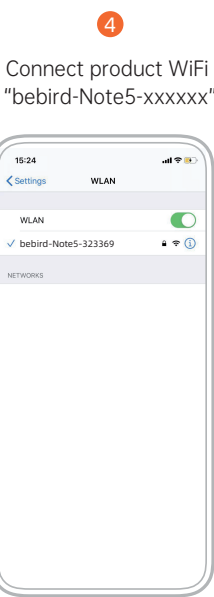

Troubleshooting

Problem	Solution
No response after startup	Please use the incidental charging cable to charge
The LED light of the camera is not on	Interrupt the connection, restart the product to connect again
The app exits quickly	Delete the existing app and scan QR to reinstall it
The indicator light blinks quickly	Please use the incidental charging cable to charge
Abnormal display in app	Interrupt the connection, restart the product to connect again
The Wifi connection was unstable and disconnected in a few seconds	Interrupt the connection, restart the product to connect again
The visual interface blur	Please use the matching cotton swab to wipe
Disorientation when in use	Please switch the left / right ear mode under the APP interface

If you still have problems with the above procedures, please contact our Customer Service Representatives directly for further assistance.

Hazardous substance

Part name	Hazardous material					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Printed Circuit Components	×	○	○	○	○	○
Cover	○	○	○	○	○	○
Plastic rubber accessory	○	○	○	○	○	○

This form is formulated in accordance with SJ/T 11364
○ : represent the hazardous material ratio is below GB/T26572-2011
× : represent the hazardous material ratio is over GB/T26572-2011


Attention

1. Please use cotton swab to wipe the lens carefully when cleaning.
2. The ear tip cover must be installed properly, not loose before use to prevent it from falling off.
3. Please observe the surrounding environment before use. Do not use it in an environment that is susceptible to impact.
4. Prohibit children from using this product alone to avoid inadvertent injury.
5. Please do not put the device inside any form of liquid.
6. Recommended to charge once a month.
7. The temperature will rise slightly when using.
8. The product temperature will rise slightly when using (from bebird laboratory data, the maximum temperature of the whole machine does not exceed about 45 °C). Please feel free to use.
9. Tweezer tips are for cerumen and foreign bodies in the ear canal only, do not use them for other purposes.

Warranty card

User information	
User name:	Telephone number:
Seller:	Shipping address:

Product Image

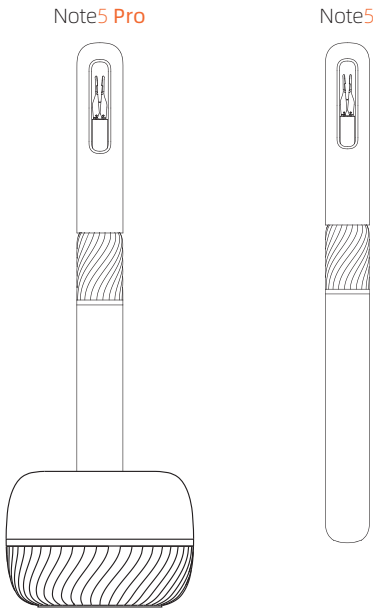


Product parameter

Brand	bebird
Product Name	Smart Visual Ear Cleaner
Optimum focal length	13.5 mm
Depth of field	10 ~ 50 mm
Gyroscope	Gimbaled Intelligent Gyroscope
Working time	About 90 minutes
Charging time	About 60 minutes
Image transmission rate	20 fps
Working temperature	-10 ~ 50°C
Input parameters	DC 5V $\pm$ 0.5A
Working frequency	2.4GHz
Network standard	IEEE 802.11b/g
Image sensor	CMOS




Product accessories diagram

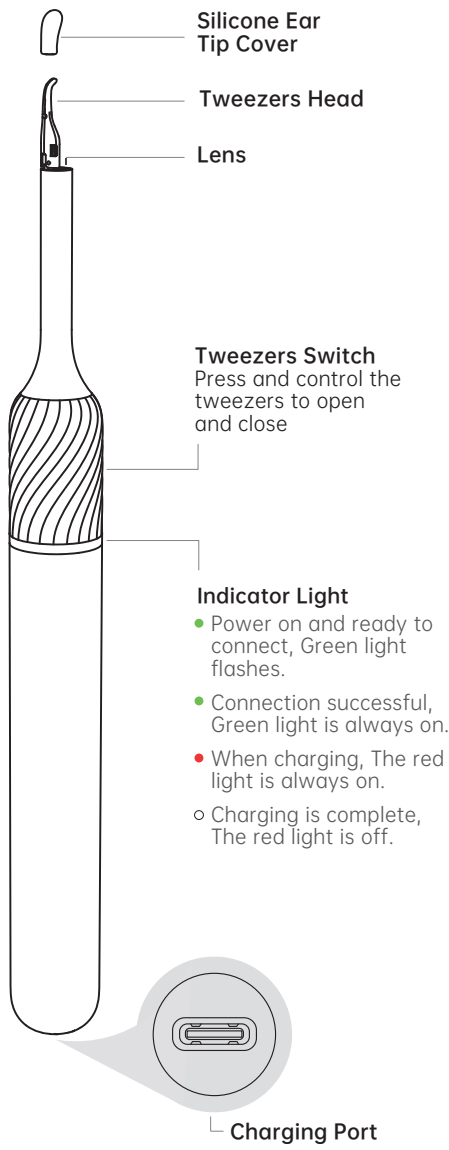

Please replace the ear tip with a new one every 2 months to avoid aging and breakage.


Note5 Accessories

The accessories of Note5 / Note5 Pro are different.
Please match the corresponding accessories according to the model purchased.

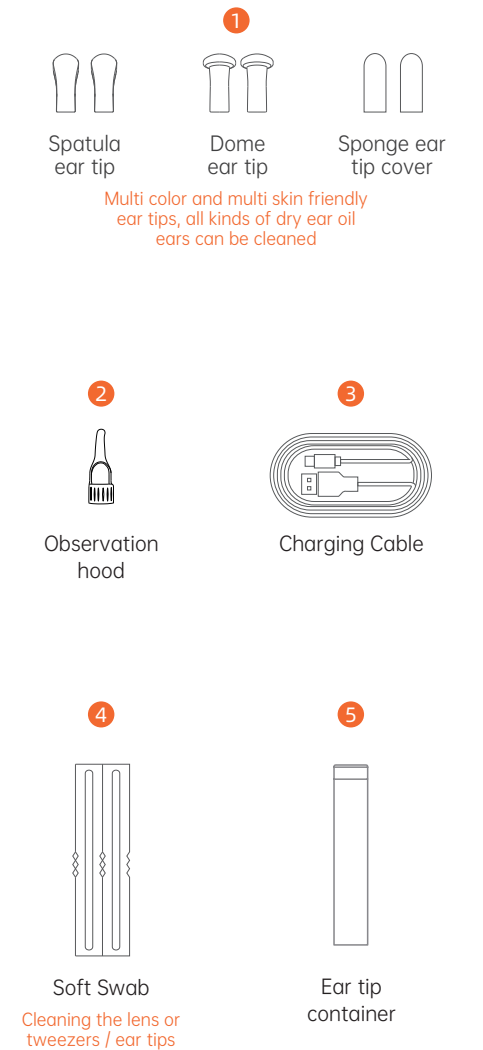

Note5 Pro Accessories

The accessories of Note5 / Note5 Pro are different.
Please match the corresponding accessories according to the model purchased.

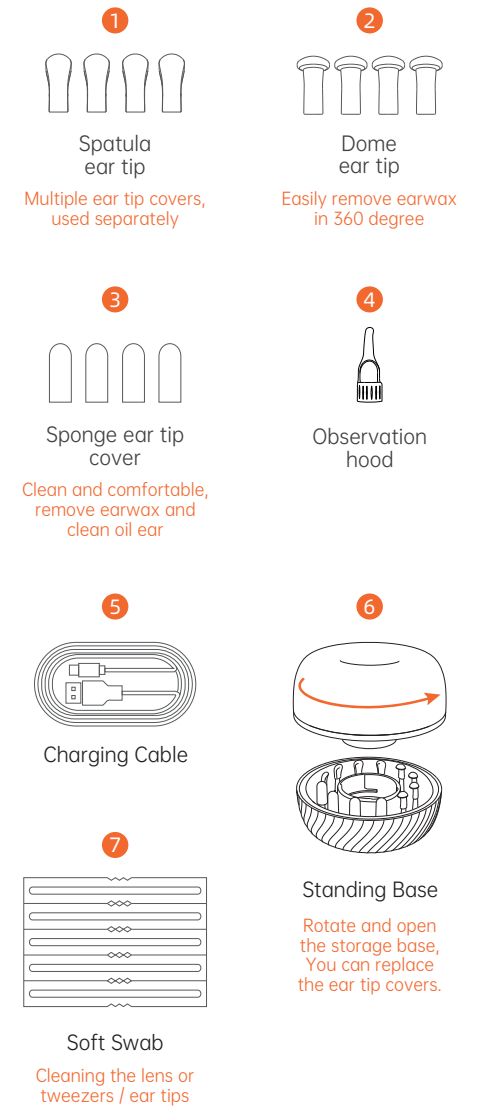


How to use the Tweezers

Hold the tweezer switch still with 2 fingers and remove the ear tip with the other hand.



How to Replace Ear Tip

Press the tweezer switch first so that the tweezers are closed. Just put the ear tip set on.

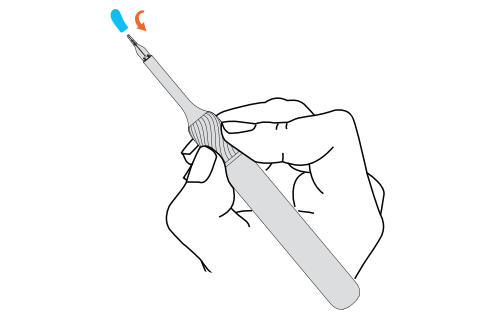

Note:
The ear tip needs to be placed as deep as possible on the tweezers to prevent it from falling off.

! If the ear tip is damaged, please stop using it immediately and contact us for replacement.